

# SIEMENS MAGNETOM Product syngo MR B17

\\USER\knee\_20min\20min\_protocols\_NYU\_27.05.09\hip\_ok\t1\_tse\_cor\_uni  
 TA: 4:20    PAT: Off    Voxel size: 0.6x0.6x3.0 mm    Rel. SNR: 1.00    SIEMENS: tse

## Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

## Geometry

Multi-slice mode	Interleaved
Series	Interleaved
-----	
Special sat.	None
-----	
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off
-----	
Tim CT mode	Off

## Routine

Slice group 1	
Slices	20
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	80 %
FoV read	180 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	670 ms
TE	12 ms
Averages	1
Concatenations	2
Filter	Distortion Corr.(2D), Elliptical filter
Coil elements	FL

## System

Body	Off
FL	On
BO1	Off
BO2	Off
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off
-----	
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
-----	
Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

## Contrast

TD	0.0 ms
MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	None
Water suppr.	None
Restore magn.	Off
-----	
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

## Resolution

Base resolution	320
Phase resolution	100 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	Off
-----	
PAT mode	None
Matrix Coil Mode	Auto (CP)
-----	
Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off

## Physio

1st Signal/Mode	None
-----	
Dark blood	Off
-----	
Resp. control	Off

## Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off

# SIEMENS MAGNETOM Product syngo MR B17

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

---

## Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	135 Hz/Px
Flow comp.	No
Allowed delay	30 s
Echo spacing	12.5 ms

---

Define	Turbo factor
Turbo factor	3
Echo trains per slice	192
RF pulse type	Normal
Gradient mode	Normal