

# syngo.via

## Keyboard Shortcuts

To maximize efficiency, *syngo.via* offers a number of shortcuts. By using keyboard shortcuts, you can, for example, use measurement tools or move through the dataset without having to switch tools.

### General

Ctrl + A	Select all segments
Ctrl + G	Select all graphics
Ctrl + Z	Undo
Ctrl + Y	Redo
Ctrl + C	Copy selection to Clipboard
Ctrl + X	Move selection to Clipboard
Ctrl + V	Paste
Ctrl + arrow keys	Move focus to next segment
Del	Delete selected graphic
Esc	End current mouse action, cancel selection
Print	Create a screenshot

### Navigation

#### 2D or MPR

Arrow key up/down	Scroll through space
Arrow key left/right	Scroll through second dimension (for example, time)

Page up/down Move focus a predefined number of images

Home	Go to first image in image stack
End	Go to last image in image stack

#### Volume

Arrow keys	Rotate volume
------------	---------------

### Tools

S	Create snapshot
R	3D reference point
Alt + R	Open Report Editor

### Measurement Tools

L	Pixel lens
D	Distance line
C	ROI circle
M	Marker

### Function keys

F1	Open Online Help
F5	Color LUT grayscale
F10	Invert color LUT
F11	Show image area only, hide other UI elements
F9	Toggle alpha blending toggling (1 <sup>st</sup> , 2 <sup>nd</sup> dataset, 50/50)

### Fused images

### Layout

T	Show/hide image text
G	Show/hide graphics
Alt + L	Open Layout Editor

### From Series Navigator to segment

Drag + left mouse button	Replace target segment content with dragged data
--------------------------	--

Drag + right mouse button	Open context menu with drag/options
---------------------------	-------------------------------------

### In Series Navigator

Double-click left mouse button	View images in floating segment
--------------------------------	---------------------------------