

# Case Study LiverLab

Stephan Kannengiesser, Ph.D.<sup>1</sup>; Radhouene Neji, Ph.D.<sup>1</sup>; Xiaodong Zhong, Ph.D.<sup>2</sup>

<sup>1</sup>Siemens Healthcare, MR Oncology Applications, Erlangen, Germany

<sup>2</sup>Siemens Healthcare, MR R&D Collaborations, Atlanta, GA, USA

Measurement data of a 22-year-old male volunteer with elevated liver fat signal, acquired on a 3T MAGNETOM Skyra system, were processed with the *syngo* MR E11A\* implementation of LiverLab\*.


The measurement program includes a regular opposed-phase / in-phase two-point Dixon protocol with water/fat separation. The First look Dixon processing first performs a liver segmentation on the water series, and then a voxel classification according to the dual-ratio Dixon signal discrimination [1]. This leads to the conclusion of fat deposition and the recommendation to run additional quantification (Fig. 1).

The *syngo* MR E11A Abdomen Dot framework uses this recommendation to implement a decision point,





through which the user can add the quantification protocols to the measurement queue. In addition, the Dot engine allows specifying an inline region-of-interest (ROI) during protocol planning. The multi-echo VIBE Dixon [2] produces inline parameter maps of fat signal percentage (proton density fat fraction, PDFF) and effective  $R2^*$ , with overlays of the liver segmentation region and pre-planned ROI. Statistics (mean, std) of PDFF and effective  $R2^*$  over the ROI and segmentation region are calculated inline, and are output as a report sheet, showing values both in text form and as color bars. Likewise, histograms are produced for PDFF and effective  $R2^*$  over the segmentation region (Fig. 2).

A HISTO (high-speed T2-corrected multi-echo single voxel spectroscopy, [3]) protocol was also run, with 5 TEs ranging from 12 to 72 ms, and a voxel of size  $3 \times 3 \times 3 \text{cm}^3$ . The HISTO inline processing integrates over the water and fat parts of the spectrum for the individual echoes, and performs a T2 relaxation correction. Results include a fat signal percentage and the  $R2$  of water ( $R2_{\text{water}}$ ), which are displayed in textual and color bar formats. A spectrum of the shortest TE and a list of quantification values for the individual echoes are added for quality control of the signal fitting. The inline analysis for this case reveals similar findings as the multi-echo VIBE Dixon (Fig. 3).

\*WIP, the product is currently under development and is not for sale in the US and other countries. Its future availability cannot be ensured.


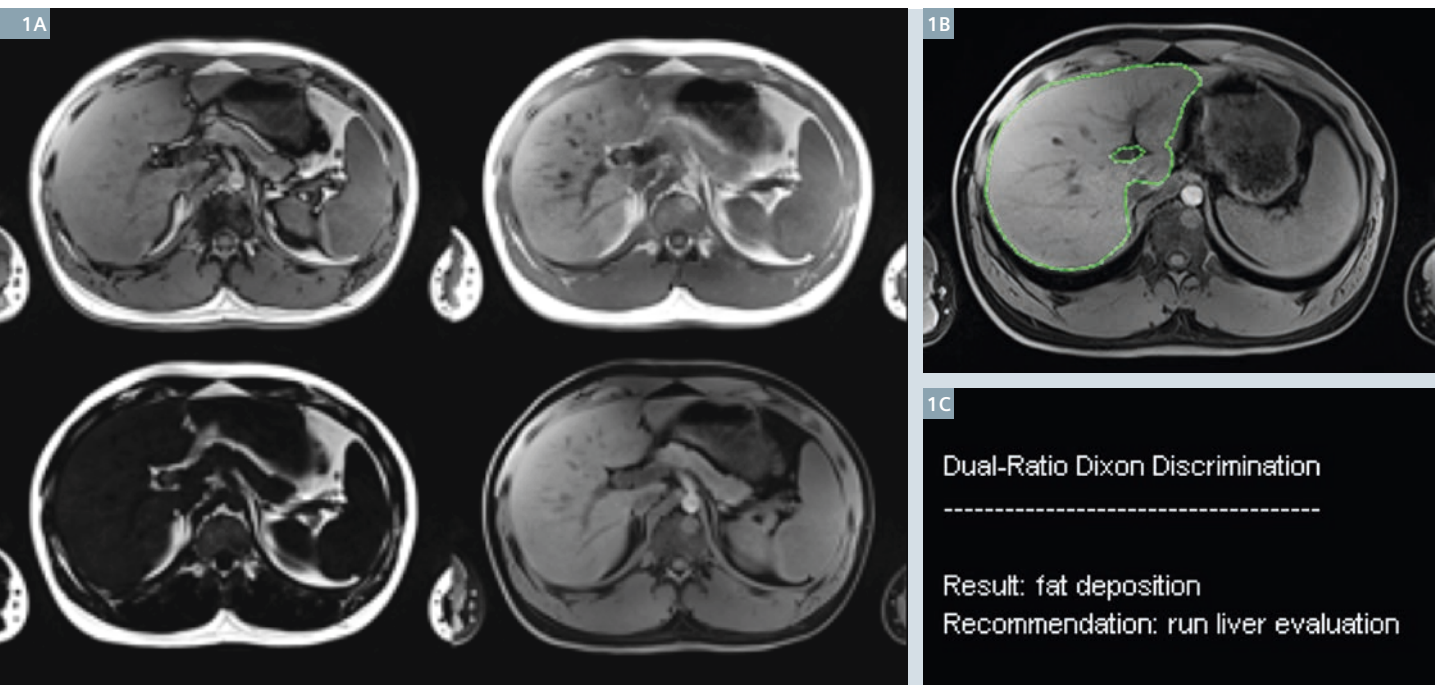


1 First look Dixon (Screening Dixon) results: (1A) opposed-phase, in-phase, fat, and water images, (1B) liver segmentation, (1C) report sheet.

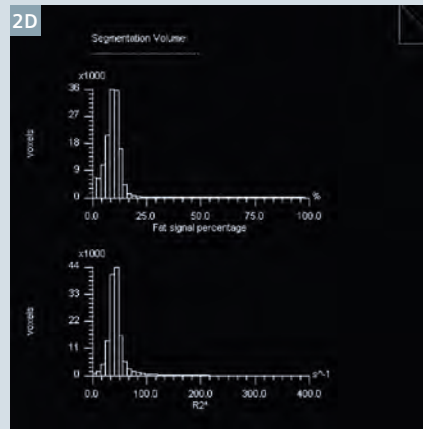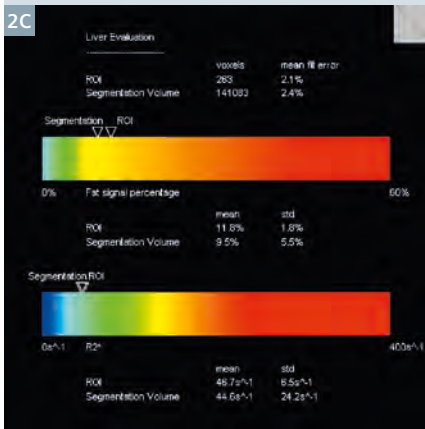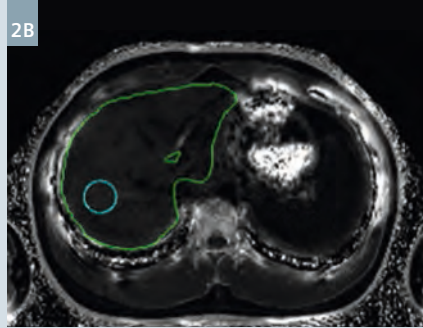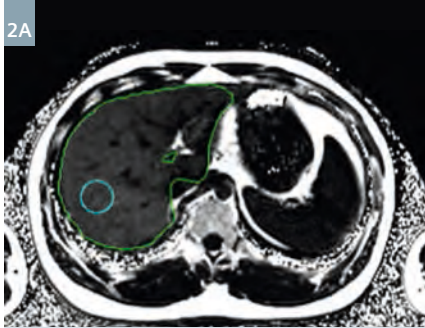



2

Multi-echo Dixon results: **(2A)** PDFF and **(2B)** effective R2\* maps with overlays of the inline liver segmentation region (green) and inline ROI (cyan). **(2C)** Inline statistics (mean, std) of PDFF and effective R2\* over ROI and segmentation region as text and color bar. The mean ROI values are fat signal percentage 11.8% and effective R2\* 46.7 s<sup>-1</sup>. **(2D)** Histograms of PDFF and effective R2\* over the segmentation region.

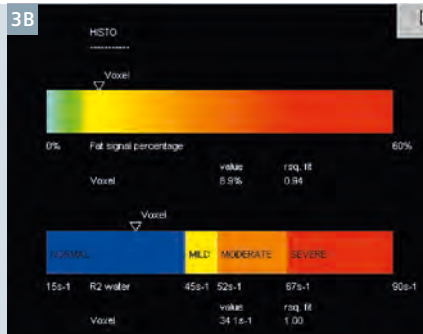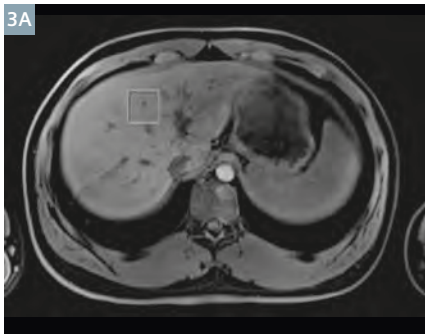

3

HISTO results: **(3A)** Voxel location (from the spectroscopy task card), **(3B)** report sheet. The values are fat signal percentage 8.9%, R2<sub>water</sub> 34.1 s<sup>-1</sup>. **(3C)** Spectrum at shortest TE, **(3D)** list of individual echo values. For comparison, a manually drawn ROI on the multi-echo Dixon parameter maps, co-localized with the HISTO voxel position shown in A), reveals PDFF 8.9%, and effective R2\* 44.3 s<sup>-1</sup>.

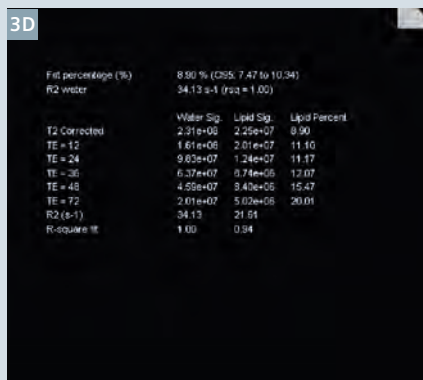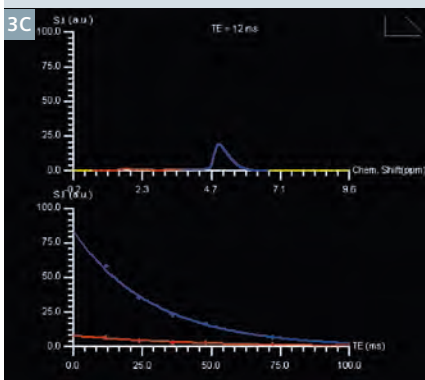

## References

- Bashir MR, Merkle EM, Smith AD, Boll DT. Hepatic MR imaging for in vivo differentiation of steatosis, iron deposition and combined storage disorder: Single-ratio in/opposed phase analysis vs. dual-ratio Dixon discrimination. *Eur J Radiol* 2012;81(2):e101-e109.
- Zhong X, Nickel MD, Kannengiesser SA, Dale BM, Kiefer B, Bashir MR. Liver fat quantification using a multi-step adaptive fitting approach with multi-echo GRE imaging. *Magn Reson Med* 2013; doi: 10.1002/mrm.25054. [Epub ahead of print]
- Pineda N, Sharma P, Xu Q, Hu X, Vos M, Martin DR. Measurement of hepatic lipid: high-speed T2-corrected multiecho acquisition at 1H MR spectroscopy – a rapid and accurate technique. *Radiology* 2009; 252:568–576.



## Contact

Stephan Kannengiesser  
Siemens Healthcare  
MR PI TIO ONCO  
Postbox 32 60  
91050 Erlangen, Germany  
Phone: +49 (1525) 4689516  
stephan.kannengiesser@siemens.com