



SARS-CoV-2 Serology Testing in the Setting of Vaccination

siemens-healthineers.com

Siemens Healthineers Position

Serology testing for SARS-CoV-2 will be beneficial and potentially even necessary in assessment of vaccine effectiveness, which will play a key role in promoting public health. Siemens Healthineers supports measuring SARS-CoV-2 IgG antibodies in relation to vaccine use for (1) establishing a threshold for protection or immunity, (2) confirming an initial neutralizing antibody response approximately 4 weeks post-vaccination, especially in those at risk for an insufficient immune response, and (3) tracking of antibody levels (at approximately 6 and 12 months) following vaccination to guide booster/revaccination efforts. An automated and scalable serology assay used for patient care in the context of vaccination should include key technical features for effective use: measurement of spike receptor-binding domain (S1-RBD)-neutralizing IgG antibodies, very high ($\geq 99.5\%$) specificity, and quantitative results.

Background

In clinical practice, quantitative antibody testing for assessing the need to vaccinate/boost is common, especially in cases such as hepatitis B vaccination, where the neutralizing surface antigen-antibody threshold associated with immunity is known.¹ In population-based studies, SARS-CoV-2 antibody testing has been shown to identify a significant percentage of the population with an immune response to the virus but undiagnosed for COVID-19.²⁻⁶ Commercially available clinical laboratory serology testing suitable for clinical practice is not expensive and can often be high-throughput, with fast turnaround and broad population access. Currently assays with very high ($\geq 99.5\%$) specificity, particularly important under conditions of low disease prevalence, will be essential to vaccination campaigns, both to identify vulnerable populations as well as assess for a successful response in large populations.^{7,8}

As learned during this pandemic for other types of SARS-CoV-2 testing, such as PCR, availability at a large and accessible scale is key to ensuring that the needs of the population can be met. While proof of antibody-

associated immunity in SARS-CoV-2 is emerging from the vaccine trials and other datasets, extensive data to date already support a role for neutralizing antibody in protecting from (or mitigating) infection.⁹⁻¹⁷

Studies from natural infections indicate significant diversity in the levels and duration of neutralizing antibody responses, with declining levels over time leading to vulnerability to reinfection.¹⁷⁻²⁶ Long COVID (or Post COVID) syndrome has been increasingly recognized, with about one-third of patients who were initially asymptomatic reported having COVID-19 symptoms weeks after diagnosis.²⁷ Consequently, serology testing is essential to identify those that may have previously been infected and to distinguish successful from suboptimal vaccine responses and detect antibody declines after natural infection.²⁸⁻³⁰ The factors influencing likelihood of a robust neutralizing antibody response are poorly defined but have been linked to immunocompetency, age, and disease severity.^{28,31,32}

Existing data indicates that detectable levels of circulating neutralizing antibody are necessary for protection, though the role of memory B-cells and/or T-cells is still under investigation.


Data from vaccine Phase II/III trials indicate that the mRNA vaccines are up to 95% effective in preventing COVID-19, and the risk of severe illness has been reported to be lowered by more than 90% in the mRNA vaccine clinical trials.^{33,34} Certain patient populations were excluded from these vaccine trials, and hence the efficacy and safety of a SARS-CoV-2 vaccine has not been established across all patient populations. To circumvent vaccine shortages, certain countries have chosen to extend the vaccine dosing interval, including the UK,³⁵ Denmark, Norway, France, and Canada.³⁶ The data regarding the safety and efficacy of the vaccines with alternative dosing schedules is currently lacking.³⁷

Considerations

Antibody-mediated immunity

Immune responses to pathogens are diverse and involve both adaptive and innate elements.³⁸⁻³⁹ Adaptive immunity is pathogen-specific and principally mediated by B- and T-cells. Humoral immunity is driven by B-cells that produce antibody (often “helped” by T-cells that secrete specific cytokines). With many pathogens, antibodies are the principal effector of protection, particularly if they can block (neutralize) viral entry.⁴⁰ Because antibody-mediated viral neutralization is a

correlate for immunity, confirmed protection in vivo associated with specific antibodies/levels must be established. A growing body of data supports the potential for neutralizing antibody to confer protection from SARS-CoV-2.^{9-17,41-43} This includes both in vitro demonstrations of antibody neutralization and in vivo evidence in a range of experimental animal models challenged with live virus. While conflicting data exists on the duration of neutralizing antibodies following SARS-CoV-2 infection, increasing datasets suggest persistence in excess of 3 months in the majority of infections, including those with mild or moderate disease but likely lesser in asymptomatic patients or those with compromised immune systems.^{17,20,21,44} More recent data from a national laboratory and clinical trials supports a sustained positivity rate of antibodies against the SARS-CoV-2 spike protein past ten months post-PCR confirmed COVID-19 infection.⁴⁵⁻⁴⁹ Antibody seropositivity persistence has been shown to last for up to two years in other coronaviruses, so epidemiological evidence of antibody duration provide longitudinal data to properly view population-level seropositivity rates, which can help shape public policy moving forward.⁵⁰ As vaccine-induced production of neutralizing antibodies proves effective, assessment of neutralizing antibody levels to identify/confirm a protective threshold will be vital in establishing broad population-based immunity. Recent modeling studies predict a significant loss in protection from SARS-CoV-2 infection following the decay of the neutralization titer over the first 250 days post immunization.⁵¹

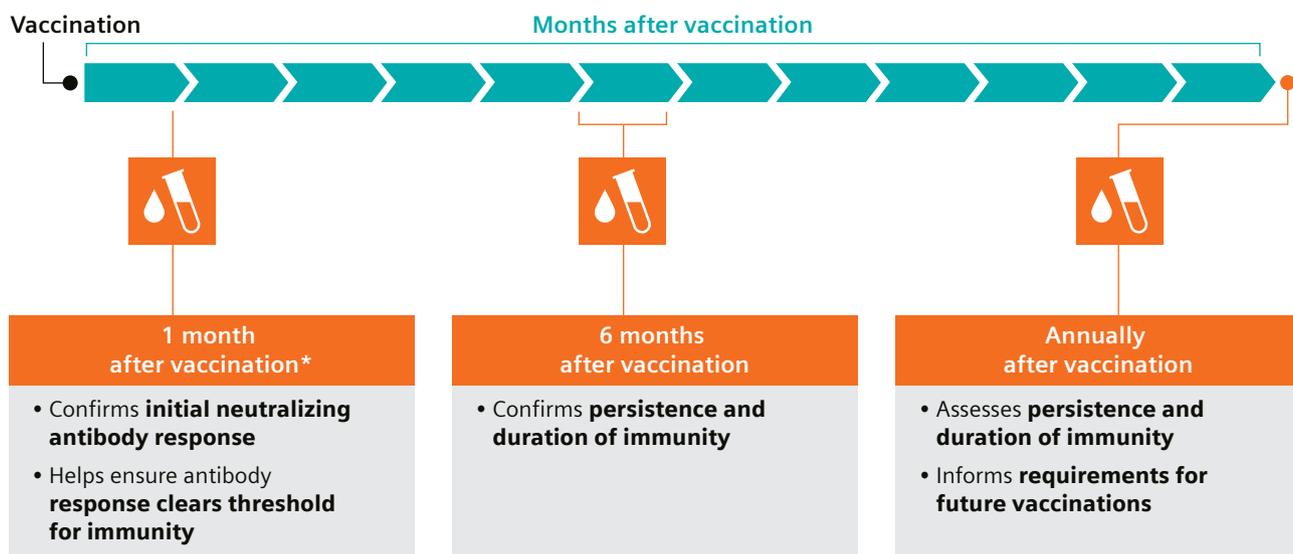

*For a 2-dose regimen, the proposed timing is after the second dose.

Figure 1. Key timepoints for serology testing to assess initial antibody immune response and duration post-vaccination.

Antibody targets and neutralization

Current commercially available SARS-CoV-2 antibody assays have diverse targets, including nucleocapsid (N) protein, whole spike (both S1 and S2 regions), S1, and S1-RBD.^{44,52-73} Robust evidence in vitro and from animal model studies supports a mechanism of viral neutralization by antibodies to the spike glycoprotein, primarily through inhibition of recognition/attachment to the ACE2 host cell receptor. While several epitope-specific neutralizing spike antibodies have been identified (in both S1 and S2), most target the S1-RBD, as these antibodies can interfere with recognition and binding to ACE2.^{9-11,17,74} Since both whole spike and S1-targeted assays include the RBD region, they can indicate, but not specifically identify, the presence of RBD-associated neutralizing antibodies. S1-RBD-specific assays are likely to prove advantageous over S1 and whole spike, especially if using a quantitative assay, as neutralizing versus binding antibodies might be expected to be enriched and therefore a better correlate to immunity. While not all antibodies to the RBD are equally neutralizing, the RBD is identified as the immunodominant source. Depletion analysis indicates an estimated ~90% of known neutralizing antibodies target epitopes within the RBD.^{10,17,41} The emergence of SARS-CoV-2 variants has led to altered sensitivity to antibody-mediated immunity through a reduction in antibody neutralization by sera from individuals with a vaccine designed against the wild type virus.^{75,76} While current data on S1-RBD vaccines may preclude the need for changes to vaccine design, the emergence of SARS-CoV-2 variants may require second generation vaccines designed with a broader set of antigenic targets to address variants.^{77,78}

Qualitative versus quantitative reporting

Qualitative SARS-CoV-2 antibody assays have a defined cut-point based on presence/absence of immune response rather than a threshold value based on antibody level and neutralization of the virus. Therefore, they only provide a “yes” or “no” indication of a response to infection. Quantitative assays of neutralizing antibody support identification of an immune threshold, above which individuals are likely to be protected and below which they are susceptible. A few IgG and total antibody quantitative assays for the spike protein (including the S1-RBD) are already commercially available.⁷⁹⁻⁸² Antibodies to SARS-CoV-2 can decline quickly and at different rates for different epitopes,^{18,19,20,22,24} so quantitation would prove salient for rapid assessment of immunity or need to boost. Quantitative testing would be a valuable tool for establishing a protective threshold, as well as initial assessment of vaccination response and monitoring of antibody levels over time when a threshold is established. Standardization of assays is required to allow comparison

of results across the different assays and can be accomplished with reference materials that are well characterized by anchoring it to a neutralization endpoint. The European Commission’s Joint Research Centre (JRC) and World Health Organization (WHO) have developed reference material for standardization.^{83,84} The First WHO International Standard (WHO 20/136) is intended to be used for calibration and harmonization of serology assays and consists of pooled convalescent plasma from recovered SARS-CoV-2 positive individuals. While its intention is for calibration, it is not suitable for harmonization or standardization, since it only contains a single value, and is not associated with neutralizing antibodies.

Serology testing for determination of immune response to vaccines

Vaccination-related testing for neutralizing antibody can be utilized at multiple time-points. Ongoing clinical trials for authorized vaccines, and vaccines in development, are utilizing serology testing for neutralizing antibody titer as a surrogate of efficacy.^{15,16,28,43,52-67} These trials are assessing neutralizing antibody immunogenicity in response to vaccine administration over time, which will be necessary to inform antibody-mediated protection. A modeling study assessing vaccine prioritization strategies demonstrated there may be value in pairing serology testing with vaccination in areas with higher SARS-CoV-2 seroprevalence for additional reductions in cumulative incidence and mortality.⁶⁸ As current vaccines require a 2-dose regimen to broadly stimulate levels of neutralizing antibody, serology testing would measure for an effective response, approximately 4 weeks post vaccination.^{43,69} Facing limited vaccine supplies, some countries have opted for an extended vaccine dosing interval (11–12 weeks) to get as many people as possible partially vaccinated. Serology testing helps evaluate how such a delay affects SARS-CoV-2 antibody levels and could inform vaccine scheduling decisions in other countries. In a cohort study in the UK, adults aged >80 years with 11–12 week intervals between doses of the Pfizer-BioNTech vaccine had 3.5-fold higher peak titers of anti-SARS-CoV-2 spike antibodies compared to adults >80 years of age given a standard three-week interval between doses.⁸⁵ Studies have suggested that people who previously had COVID-19 may get a strong immune response from only one dose of these vaccines.⁸⁶⁻⁸⁸ Serology testing can identify previously infected individuals and evaluate antibody response following a one or two dose regimen. France has formalized a vaccine policy for people who have had and recovered from SARS-CoV-2 infection and announced that vaccine centers will be equipped with antibody tests to test everyone before they are vaccinated.⁸⁹

For logistical convenience, in many settings serology testing can be undertaken during the same visit as the second vaccine dose administration. Quantitative periodic antibody testing post-vaccination, after approximately 6 and 12 months, would assure a sustained antibody response at sufficient levels for virus neutralization and guide booster shot prioritization (Figure 1). The timing of appropriate serology testing would be optimized and refined as needed and may differ between individuals based on the strength of their initial response to vaccination. Vaccine manufacturers have indicated a likely need for booster doses to combat variants and prolong protection against wild-type SARS-CoV-2.^{90,91} Periodic antibody testing can identify at risk populations that would benefit from getting a booster dose and inform decision-making to guide prioritization strategies for booster dose administration in various populations.

A serology-defined threshold (from either natural infection or vaccination) remains a key need and this periodic testing would offer additional data on antibody response patterns to determine optimal serology testing utilization. Longer time-frame quantitative testing for waning levels of protective antibody, such as through annual testing, would inform the need to revaccinate/boost if SARS-CoV-2 becomes a seasonal disease.

Vaccines and efficacy in current clinical trials

In phase 3 vaccine trials, protection from disease, i.e., immunity, has been demonstrated relative to the placebo group despite a finite incidence of infection in the vaccinated subjects. A vaccine could achieve statistical significance for the primary endpoint for protection from disease despite significant incidence of disease in the vaccinated group (example FDA accepted primary endpoint >50%).⁹² Even with high efficacy, a proportion of those inoculated would not have protection from disease. Assessment for seroconversion failure or declining levels in the vaccinated but susceptible population is a critical parameter with implications for patient care, population management, and public policy.^{5,93} Data from initial vaccine trials is limited to certain populations and exposure patterns. Additional data on antibody response and duration will be needed to help inform vaccine efficacy in larger, more-diverse populations to determine appropriate use in the context of variables such as vaccine design/manufacturer, ethnicity, level of viral load exposure, chronic diseases, and individual immune system strength.⁹⁴ All vaccines in use or development published on to date include or are based solely on the spike protein, with spike- or RBD-specific antibodies serving as a surrogate of efficacy along with elements of the cellular response. In this scenario, natural infection can be monitored by testing

for antibody to the N protein. However, testing for quantitative S1-RBD antibodies would be the preferred method to assess levels relative to susceptibility following vaccination due to their correlation to neutralization and protection. Additional data on vaccine use and antibody response in already-seropositive patients is needed to determine response patterns in a more-diverse antibody population. Following the identification of multiple SARS-COV2 variant strains, additional data on the ability of antibodies developed from vaccines to infer protection against these strains would be necessary.⁷⁵⁻⁷⁸

Vaccine response in at-risk populations

Initially, additional data on duration of antibody-mediated protection is needed across populations, and in the long-term testing may be focused on populations with known risk of insufficient immune response such as the elderly.^{95,96} Vaccine clinical trial data currently supports neutralizing antibody persistence for some vaccine recipients for at least 6 months, with an average half-life of 52 days.⁹⁶ The timing of appropriate serology testing would be optimized and refined based on the accumulation of evidence. In 2013, researchers from the CDC estimated that the prevalence of immunosuppression among adults in the United States was 2.7%.⁹⁷ This patient population has been excluded from the vaccine trials. Data for other vaccines, such as regular-dose influenza vaccine, show a reduced response in solid organ transplant recipients when compared with the general population.^{98,99} Studies assessing the immune response of SARS-CoV-2 vaccinated individuals have relied on the measurement of the level of antibody response (or lack thereof) in immunocompromised individuals in comparison to healthy individuals as a measure of vaccine efficacy.^{100,101} Studies in various patient populations such as solid organ transplant recipients,¹⁰²⁻¹⁰⁶ dialysis,¹⁰⁷⁻¹¹¹ hematologic malignancies,^{100,112-113} multiple myeloma,^{114,115} inflammatory bowel disease receiving biologic therapies,^{116,117} and treatments with immunosuppressive medications,¹¹⁸ have shown a varied antibody response following vaccination. Studies on duration of antibody response in these populations have been lacking. Quantitative serology testing would allow physicians to assess the presence, levels, and duration of antibody response following SARS-CoV-2 vaccination, and help guide the need for additional or higher doses as with other vaccines in solid organ transplant recipients.⁹⁸ With the rollout of vaccinations to pediatric population, serology testing to assess immune response in more common immunocompromised populations, such as acute lymphoblastic leukemia and sickle cell disease, and populations on immunomodulating drugs will be needed.¹¹⁹

Summary

To enable an effective vaccination strategy, Siemens Healthineers advocates the use of automated SARS-CoV-2 serology testing to help confirm efficacy.

Serology assays should have the appropriate characteristics for assessment of vaccine response:

- Quantitative results
- S1-RBD-neutralizing IgG antibody detection
- Very high ($\geq 99.5\%$) specificity

Serology testing can inform vaccination utilization and status of protection at multiple junctures, especially in at-risk populations:

- Post-vaccination initial response after approximately 4 weeks
- 6 and 12 months to guide decision making regarding the need for booster doses

Additionally, quantitative neutralizing-antibody testing could support determination of an antibody threshold for immunity/susceptibility to SARS-CoV-2 and provide critical data needed to understand vaccine-facilitated antibody response and duration in populations not included in initial vaccine trials. Serology is a scientifically sound, cost-effective surrogate for vaccine efficacy and able to meet high-volume testing needs. Ensuring the effectiveness of vaccines will play a key role in promoting public health, including assessing sufficient and durable protection.

References

- Walayat S, et al. Recent advances in vaccination of non-responders to standard dose hepatitis B virus vaccine. *World J Hepatol.* 2015 October 28;7(24):2503-9. <https://doi.org/10.4254/wjh.v7.i24.2503>
- Stadlbauer D, et al. Repeated cross-sectional sero-monitoring of SARS-CoV-2 in New York City. *Nature.* 2020. <https://doi.org/10.1038/s41586-020-2912-6>
- Dingens AS. Seroprevalence of SARS-CoV-2 among children visiting a hospital during the initial Seattle outbreak. *Nature Communications.* 2020. 11:4378. <https://doi.org/10.1038/s41467-020-18178-1>
- Gudbjartsson DF, et al. Humoral immune response to SARS-CoV-2 in Iceland. *N Engl J Med.* 2020;383:1724-34. DOI: 10.1056/NEJMoa2026116
- Alter G, Seder R. The power of antibody-based surveillance. *N Engl J Med.* 2020; 383:1782-4. <https://doi.org/10.1056/NEJMe2028079>
- Perico, L. et al. COVID-19 and Lombardy: TESTING the impact of the first wave of the pandemic. *EBioMedicine.* 2020. <https://doi.org/10.1016/j.ebiom.2020.103069>
- <https://www.cdc.gov/coronavirus/2019-ncov/lab/resources/antibody-tests-guidelines.html>. Accessed November 6, 2020.
- <https://www.idsociety.org/COVID19guidelines/serology>. Accessed November 6, 2020.
- Rodda, L.B. et al. Functional SARS-CoV-2-specific immune memory persists after mild COVID-19. *Cell* vol. 184,1 (2021): 169-183.e17. <https://doi.org/10.1016/j.cell.2020.11.029>
- Piccoli, L. et al. Mapping Neutralizing and Immunodominant Sites on the SARS-CoV-2 Spike Receptor-Binding Domain by Structure-Guided High-Resolution Serology. *Cell* vol. 183,4 (2020): 1024-1042.e21. <https://doi.org/10.1016/j.cell.2020.09.037>
- Premkumar, L. et al. The receptor binding domain of the viral spike protein is an immunodominant and highly specific target of antibodies in SARS-CoV-2 patients. *Science immunology* vol. 5,48 (2020): eabc8413. <https://doi.org/10.1126/sciimmunol.abc8413>
- Wang, H. et al. Development of an inactivated vaccine candidate, BBIBP-CorV, with potent protection against SARSCoV-2. 2020. *Cell.* 182;713-21. <https://doi.org/10.1016/j.cell.2020.06.008>
- Addetia, A., et al. Neutralizing antibodies correlate with protection from SARS-CoV-2 in humans during a fishery vessel outbreak with a high attack rate. 2020. *J Clin Microbiol.* 58:e02107-20. <https://doi.org/10.1128/JCM.02107-20>.
- Corbett KS, et al. Evaluation of the mRNA-1273 vaccine against SARS-CoV-2 in nonhuman primates. *N Engl J Med.* 2020 Jul 28;NEJMoa2024671. <https://doi.org/10.1056/NEJMoa2024671>.
- Walsh, E.E. et al. Safety and Immunogenicity of Two RNA-Based Covid-19 Vaccine Candidates. *N Engl J Med.* 2020 383:2439-50. <https://doi.org/10.1056/nejmoa2027906>.
- Mercado NB, et al. Single-shot Ad26 vaccine protects against SARS-CoV-2 in rhesus macaques. *Nature* vol. 586,7830 (2020): 583-588. <https://doi.org/10.1038/s41586-020-2607-z>
- Wajnberg A, et al. Robust neutralizing antibodies to SARS-CoV-2 infection persist for months. *Science* vol. 370,6521 (2020): 1227-1230. <https://doi.org/10.1126/science.abd7728>.
- Long, Q. et al. Clinical and immunological assessment of asymptomatic SARS-CoV-2 infections. *Nat Med.* 2020;26:1200-4. <https://doi.org/10.1038/s41591-020-0965-6>
- Seow, J. et al. Longitudinal observation and decline of neutralizing antibody responses in the three months following SARS-CoV-2 infection in humans. *Nature microbiology* vol. 5,12 (2020):1598-1607. <https://doi.org/10.1038/s41564-020-00813-8>
- Muecksch F, et al. Longitudinal Serological Analysis and Neutralizing Antibody Levels in Coronavirus Disease 2019 Convalescent Patients, *The Journal of Infectious Diseases,* Volume 223, Issue 3, 1 February 2021, Pages 389–398. <https://doi.org/10.1093/infdis/jiaa659>
- Ripperger TJ, et al. Orthogonal SARS-CoV-2 Serological Assays Enable Surveillance of Low-Prevalence Communities and Reveal Durable Humoral Immunity. *Immunity* vol. 53,5 (2020): 925-933.e4. <https://doi.org/10.1016/j.immuni.2020.10.004>
- Ibarrondo, F.J. et al., et al. Rapid decay of anti-SARS-CoV-2 antibodies in persons with mild Covid-19. *N Engl J Med.*2020;383:1085-7. <https://doi.org/10.1056/NEJMc2025179>
- Wang Y, et al. Kinetics of viral load and antibody response in relation to COVID-19 severity. *J Clin Invest.* 2020;130(10):5235-44. <https://doi.org/10.1172/JCI138759>.
- Grandjean L, et al. Humoral response dynamics following infection with SARS-CoV-2. *medRxiv.* 2020.07.16.20155663. <https://doi.org/10.1101/2020.07.16.20155663>
- Fenwick, C. et al. Changes in SARS-CoV-2 Spike versus Nucleoprotein Antibody Responses Impact the Estimates of Infections in Population-Based Seroprevalence Studies." *Journal of virology* vol. 95,3 e01828-20. 13 Jan. 2021. <https://doi.org/10.1128/JVI.01828-20>
- Harvala, H. et al. Convalescent plasma treatment for SARS-CoV-2 infection: analysis of the first 436 donors in England, 22 April to 12 May 2020. *Euro Surveill.* 2020;25(28):2001260. <https://doi.org/10.2807/1560-7917.ES.2020.25.28.2001260>
- Huang, Y. et al. COVID Symptoms, Symptom Clusters, and Predictors for Becoming a Long-Hauler: Looking for Clarity in the Haze of the Pandemic. *medRxiv:* 2021.03.03.21252086. 5 Mar. 2021, <https://doi.org/10.1101/2021.03.03.21252086>
- Poland GA, et al. SARS-CoV-2 immunity: review and applications to phase 3 vaccine candidates. Published online October 13, 2020. [https://doi.org/10.1016/S0140-6736\(20\)32137-1](https://doi.org/10.1016/S0140-6736(20)32137-1)
- Erasmus JH, et al. An alphavirus-derived replicon RNA vaccine induces SARS-CoV-2 neutralizing antibody and T cell responses in mice and nonhuman primates. *Sci Transl Med.* 2020. <https://doi.org/10.1126/scitranslmed.2020.06.008>
- Jeyanathan M, et al. Immunological considerations for COVID-19 vaccine strategies. *Nat Rev Immunol.* 2020;20:615-32. <https://doi.org/10.1038/s41577-020-00434-6>
- Chen Y, et al. Aging in COVID-19: vulnerability, immunity and intervention. *Ageing Res Rev.* 2020 Oct 31:101205. <https://doi.org/10.1016/j.arr.2020.101205>
- Poonia, B. and Kottlilil, S. Immune correlates of COVID-19 control. *Front Immunol.* 2020;11:569611. <https://doi.org/10.3389/fimmu.2020.569611> <https://doi.org/10.1016/j.jaci.2009.12.980>
- Baden, L.R. et al. Efficacy and Safety of the mRNA-1273 SARS CoV-2 Vaccine. *N Engl J Med.* 2021;384(5):403-416. <https://doi.org/10.1056/NEJMoa2035389>
- Polack, F.P. et al. Safety and Efficacy of the BNT162b2 mRNA Covid-19 Vaccine. *N Engl J Med.* 2020;383(27):2603-2615. <https://doi.org/10.1056/NEJMoa2034577>
- NHS. Letter to chief executives of all NHS trusts and foundation trusts. 30 Dec 2020. <https://www.england.nhs.uk/coronavirus/wp-content/uploads/sites/52/2020/12/C0994-System-letter-COVID-19-vaccination-deployment-planning-30-December-2020.pdf>. Accessed June 2, 2021
- <https://www.straitstimes.com/world/several-countries-delaying-second-covid-19-vaccine-dose> accessed June 2, 2021
- Iacobucci, G. et al. Covid-19 vaccination: What's the evidence for extending the dosing interval? *BMJ* 2021;372:n18 <https://doi.org/10.1136/bmj.n18>

38. Chaplin DD. Overview of the immune response. *J Allergy Clin Immunol*. 2010 Feb;125(2 Suppl 2):S3-23. <https://doi.org/10.1016/j.jaci.2010.02.001>.
39. Marshall, J.S. et al. An introduction to immunology and immunopathology. *Allergy Asthma Clin Immunol*. 2018 Sep 12;14(Suppl 2):49. <https://doi.org/10.1186/s13223-018-0278-1>
40. Van Blargan LA, et al. Deconstructing the antiviral neutralizing-antibody response: implications for vaccine development and immunity. *Microbiology and Molecular Biology Reviews*. 2016 Oct; 80(4):989-1010. <https://doi.org/10.1128/MMBR.00024-15>
41. Chen X, et al. Disease severity dictates SARS-CoV-2-specific neutralizing antibody responses in COVID-19. *Sig Transduct Target Ther*. 2020;5:180. <https://doi.org/10.1038/s41392-020-00301-9>
42. Hassan AO, et al. A SARS-CoV-2 infection model in mice demonstrates protection by neutralizing antibodies. *Cell*. 2020 Aug 6;182:744-53. <https://doi.org/10.1016/j.cell.2020.06.011>
43. Krammer F. SARS-CoV-2 vaccines in development. *Nature*. 2020;586:516-27. <https://doi.org/10.1038/s41586-020-2798-3>
44. Figueiredo-Campos P, et al. Seroprevalence of anti-SARS-CoV-2 antibodies in COVID-19 patients and healthy volunteers up to 6 months post disease onset. *Eur J Immunol*. 2020;00:1-16. <https://doi.org/10.1002/eji.202048970>.
45. Alfego, D. et al., A population-based analysis of the longevity of SARS-CoV-2 antibody seropositivity in the United States, *EclinicalMedicine* (2021). <https://doi.org/10.1016/j.eclinm.2021.100902>
46. He, Zhenyu et al. Seroprevalence and humoral immune durability of anti-SARS-CoV-2 antibodies in Wuhan, China: a longitudinal, population-level, cross-sectional study. *Lancet* (London, England) vol. 397,10279 (2021): 1075-1084. [https://doi.org/10.1016/S0140-6736\(21\)00238-5](https://doi.org/10.1016/S0140-6736(21)00238-5)
47. Glüick, V. et al. SARS-CoV-2-directed antibodies persist for more than six months in a cohort with mild to moderate COVID-19. *Infection* (2021). <https://doi.org/10.1007/s15010-021-01598-6>
48. Den Hartog, G. et al. Persistence of antibodies to SARS-CoV-2 in relation to symptoms in a nationwide prospective study, *Clinical Infectious Diseases*, 2021;, ciab172, <https://doi.org/10.1093/cid/ciab172>
49. L'Huillier, Arnaud G et al. Antibody persistence in the first 6 months following SARS-CoV-2 infection among hospital workers: a prospective longitudinal study. *Clinical microbiology and infection*, vol. 27,5 784.e1–784.e8. 20 Jan. 2021, <https://doi.org/10.1016/j.cmi.2021.01.005>
50. Huang, A.T. et al. A systematic review of antibody mediated immunity to coronaviruses: kinetics, correlates of protection, and association with severity. *Nat Commun* 11, 4704 (2020). <https://doi.org/10.1038/s41467-020-18450-4>
51. Khoury, D. S. et al. Neutralizing antibody levels are highly predictive of immune protection from symptomatic SARS-CoV-2 infection. *Nature medicine*, 10.1038/s41591-021-01377-8. 17 May. 2021, <https://doi.org/10.1038/s41591-021-01377-8>
52. Xia, S. et al. Effect of an inactivated vaccine against SARS-CoV-2 on safety and immunogenicity outcomes: interim analysis of 2 randomized clinical trials. *JAMA* vol. 324,10 (2020): 951-960. <https://doi.org/10.1001/jama.2020.15543>
53. Folegatti PM, et al. Safety and immunogenicity of the ChAdOx1 nCoV-19 vaccine against SARS-CoV-2: a preliminary report of a phase ½, single-blind, randomized controlled trial. *Lancet*. 2020;396:467-78. [https://doi.org/10.1016/S0140-6736\(20\)31604-4](https://doi.org/10.1016/S0140-6736(20)31604-4)
54. Zhu, F.C. et al. Safety, tolerability, and immunogenicity of a recombinant adenovirus type-5 vectored COVID-19 vaccine: a dose-escalation, open-label, non-randomised, first-in-human trial. *Lancet*. 2020;395:1845-54. [https://doi.org/10.1016/S0140-6736\(20\)31208-3](https://doi.org/10.1016/S0140-6736(20)31208-3)
55. Zhu, F.C. et al. Immunogenicity and safety of a recombinant adenovirus type-5-vectored COVID-19 vaccine in healthy adults aged 18 years or older: a randomized, double-blind, placebo-controlled, phase 2 trial. *Lancet*. 2020;396(10249):479-88. [https://doi.org/10.1016/S0140-6736\(20\)31605-6](https://doi.org/10.1016/S0140-6736(20)31605-6)
56. Logunov DY, et al. Safety and immunogenicity of an rAd26 and rAd5 vector-based heterologous prime-boost COVID-19 vaccine in two formulations: two open, non-randomised phase ½ studies from Russia. *Lancet*. 2020;396:887-97. [https://doi.org/10.1016/S0140-6736\(20\)31866-3](https://doi.org/10.1016/S0140-6736(20)31866-3)
57. Jackson LA, et al. An mRNA vaccine against SARS-CoV-2 — preliminary report. *N Engl J Med*. 2020 Jul 14;383:1920-31. <https://doi.org/10.1056/NEJMoa2022483>.
58. Keech C, et al. Phase 1–2 trial of a SARS-CoV-2 recombinant spike protein nanoparticle vaccine. *N Engl J Med* 2020; 383:2320-2332. <https://doi.org/10.1056/NEJMoa2026920>
59. Zhang, Y.J. et al. Immunogenicity and safety of a SARS-CoV-2 inactivated vaccine in healthy adults aged 18-59 years: report of the randomized, double-blind, and placebo-controlled phase 2 clinical trial. *medRxiv*. 2020.07.31.20161216; <https://doi.org/10.1101/2020.07.31.20161216>
60. Hotez, P.J. et al. COVID-19 vaccines: neutralizing antibodies and the alum advantage. *Nat Rev Immunol*. 2020;20:399-400. <https://doi.org/10.1038/s41577-020-0358-6>
61. Dagotto G, et al. Approaches and challenges in SARS-CoV-2 vaccine development. *Cell Host Microbe*. 2020 Sep 9;28(3):364-70. Published online 2020 Aug 10. <https://doi.org/10.1016/j.chom.2020.08.002>
62. Yu J, et al. DNA vaccine protection against SARS-CoV-2 in rhesus macaques. *Science*. 2020 Aug 14;369(6505):806-11. <https://doi.org/10.1126/science.abc6284>. Epub 2020 May 20.
63. van Doremalen N, et al. ChAdOx1 nCoV-19 vaccine prevents SARS-CoV-2 pneumonia in rhesus macaques. *Nature*. 2020;586:578-82. <https://doi.org/10.1038/s41586-020-2608-y>
64. Gao Q, et al. Development of an inactivated vaccine candidate for SARS-CoV-2. *Science*. 2020;369:77-81. <https://doi.org/10.1126/science.abc1932>.
65. Mulligan MJ, et al. Phase I/II study of COVID-19 RNA vaccine BNT162b1 in adults. *Nature*. 2020;586:589-93. <https://doi.org/10.1038/s41586-020-2639-4>
66. Sahin U, et al. COVID-19 vaccine BNT162b1 elicits human antibody and TH1 T cell responses. *Nature*. 2020;586:594-9. <https://doi.org/10.1038/s41586-020-2814-7>
67. Bos R, et al. Ad26 vector-based COVID-19 vaccine encoding a prefusion-stabilized SARS-CoV-2 spike immunogen induces potent humoral and cellular immune responses. *NPJ Vaccines*. 2020 Sep 28;5:91. <https://doi.org/10.1038/s41541-020-00243-x>.
68. Bubar, K.M. et al. Model-informed COVID-19 vaccine prioritization strategies by age and serostatus. *Science* vol. 371,6532 (2021): 916-921. <https://doi.org/10.1126/science.abe6959>
69. Gaebler, C. & Nussenzweig, M.C. All eyes on the hurdle race for a SARS-CoV-2 vaccine. *Nature* vol. 586,7830 (2020): 501-502. <https://doi.org/10.1038/d41586-020-02926-w>
70. Traub, M.A. et al. Comparison of eight commercial, high-throughput, automated or ELISA assays detecting SARS-CoV-2 IgG or total antibody. *Journal of Clinical Virology*. 2020;132:104613. <https://doi.org/10.1016/j.jcv.2020.104613>
71. Houlihan, C.F. and Beale R. The complexities of SARS-CoV-2 serology. 2020 Sep 23. [https://doi.org/10.1016/S1473-3099\(20\)30699-X](https://doi.org/10.1016/S1473-3099(20)30699-X)
72. National SARS-CoV-2 Serology Assay Evaluation Group. Performance characteristics of five immunoassays for SARS-CoV-2: a head-to-head benchmark comparison. *Lancet Infect Dis*. Published online 2020 Sep 23. [https://doi.org/10.1016/S1473-3099\(20\)30634-4](https://doi.org/10.1016/S1473-3099(20)30634-4)

References (continued)

73. Rosadas C, et al. Testing for responses to the wrong SARS-CoV-2 antigen? *Lancet*. 2020 Sep 5; 396(10252):E23. [https://doi.org/10.1016/S0140-6736\(20\)31830-4](https://doi.org/10.1016/S0140-6736(20)31830-4).
74. Iyer, A.S. Dynamics and significance of the antibody response to SARS-CoV-2. *medRxiv*. 2020.07.18.20155374. DOI:10.1101/2020.07.18.20155374
75. Wang, P. et al. Increased Resistance of SARS-CoV-2 Variant P.1 to Antibody Neutralization. *Cell Host & Microbe*, Volume 29, Issue 5, 2021, Pages 747-751.e4. <https://doi.org/10.1016/j.chom.2021.04.007>
76. Wibmer, C.K. et al. SARS-CoV-2 501Y.V2 escapes neutralization by South African COVID-19 donor plasma. *Nat Med* 27, 622–625 (2021). <https://doi.org/10.1038/s41591-021-01285-x>
77. Walensky, R.P. et al. SARS-CoV-2 Variants of Concern in the United States—Challenges and Opportunities. *JAMA*. 2021;325(11):1037–1038. <https://doi.org/10.1001/jama.2021.2294>
78. Mahase E. Covid-19: Where are we on vaccines and variants? *BMJ* 2021; 372 :n597 <https://doi.org/10.1136/bmj.n597>
79. <https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/eua-authorized-serology-test-performance>. Accessed November 12, 2020.
80. <https://www.siemens-healthineers.com/press-room/press-releases/covid-19-antibody-igg.html>. Accessed November 12, 2020.
81. <https://www.roche.com/media/releases/med-cor-2020-09-18b.htm>. Accessed November 12, 2020.
82. <https://newsnetwork.mayoclinic.org/discussion/mayo-clinic-launches-neutralizing-antibody-test-to-advance-covid-19-therapies/>. Accessed November 12, 2020.
83. WHO/BS.2020.2403 Establishment of the WHO International Standard and Reference Panel for anti-SARS-CoV-2 antibody: <https://www.who.int/publications/m/item/WHO-BS-2020.2403> (accessed June 9, 2021).
84. European Commission, Joint Research Centre, Directorate F – Health, Consumers and Reference Materials, Reference Material Product Information Sheet. EURM-017. <http://www.https://crm.jrc.ec.europa.eu/p/EURM-017>, 2020 (accessed 5 February, 2021).
85. Parry, H.M. et al. (May 17, 2021). Extended Interval BNT162b2 Vaccination Enhances Peak Antibody Generation in Older People. *medRxiv* 2021.05.15.21257017 <https://doi.org/10.1101/2021.05.15.21257017>
86. Krammer, F. et al. Antibody Responses in Seropositive Persons after a Single Dose of SARS-CoV-2 mRNA Vaccine. *N Engl J Med*. 2021 Apr 8;384(14):1372-1374. <https://doi.org/10.1056/NEJMc2101667>
87. Saadat, S. et al. Binding and Neutralization Antibody Titers After a Single Vaccine Dose in Health Care Workers Previously Infected With SARS-CoV-2. *JAMA*. 2021;325(14):1467–1469. <https://doi.org/10.1001/jama.2021.3341>
88. Ebinger, J.E. et al. Antibody responses to the BNT162b2 mRNA vaccine in individuals previously infected with SARS-CoV-2. *Nat Med* (2021). <https://doi.org/10.1038/s41591-021-01325-6>
89. <https://www.thelocal.fr/20210604/french-health-minister-anyone-who-has-had-covid-only-needs-one-vaccine-dose/> (accessed June 9, 2021)
90. <https://www.cnn.com/2021/04/15/pfizer-ceo-says-third-covid-vaccine-dose-likely-needed-within-12-months.html> accessed June 2, 2021
91. <https://www.cbsnews.com/news/moderna-covid-vaccine-boostershots/> accessed June 2, 2021
92. FDA. Development and licensure of vaccines to prevent COVID-19. Guidance for industry. <https://www.fda.gov/media/139638/download>. Accessed November 8, 2020.
93. Corbett KS, et al. SARS-CoV-2 mRNA vaccine design enabled by prototype pathogen preparedness. *Nature*. 2020. <https://doi.org/10.1038/s41586-020-2622-0>
94. Dagan, N. et al. BNT162b2 mRNA Covid-19 Vaccine in a Nationwide Mass Vaccination Setting. *The New England journal of medicine* vol. 384,15 (2021): 1412-1423. <https://doi.org/10.1056/NEJMoa2101765>
95. Hansen, C.H. et al. Assessment of protection against reinfection with SARS-CoV-2 among 4 million PCR-tested individuals in Denmark in 2020: a population-level observational study. *Lancet* vol. 397,10280 (2021): 1204-1212. [https://doi.org/10.1016/S0140-6736\(21\)00575-4](https://doi.org/10.1016/S0140-6736(21)00575-4)
96. Doria-Rose, N. et al. Antibody Persistence through 6 Months after the Second Dose of mRNA-1273 Vaccine for Covid-19. *The New England journal of medicine* vol. 384,23 (2021): 2259-2261. <https://doi.org/10.1056/NEJMc210391>
97. Harpaz, R. et al. Prevalence of Immunosuppression Among US Adults, 2013. *JAMA*. 2016;316(23):2547–2548. <https://doi.org/10.1001/jama.2016.16477>
98. Natori, Y. et al. A Double-Blind, Randomized Trial of High-Dose vs Standard-Dose Influenza Vaccine in Adult Solid-Organ Transplant Recipients. *Clin Infect Dis*. 2018 May 17;66(11):1698-1704. <https://doi.org/10.1093/cid/cix1082>
99. Chong, P.P. et al. A Systematic Review of Safety and Immunogenicity of Influenza Vaccination Strategies in Solid Organ Transplant Recipients. *Clin Infect Dis*. 2018 May 17;66(11):1802-1811. <https://doi.org/10.1093/cid/cix1081>
100. Agha, M. et al. (April 7, 2021) Suboptimal response to COVID-19 mRNA vaccines in hematologic malignancies patients. *medRxiv* 2021 Apr 7:2021.04.06.21254949 <https://doi.org/10.1101/2021.04.06.21254949>
101. Massarweh, A. et al. Evaluation of Seropositivity Following BNT162b2 Messenger RNA Vaccination for SARS-CoV-2 in Patients Undergoing Treatment for Cancer. *JAMA Oncol*. Published online May 28, 2021. <https://doi.org/10.1001/jamaoncol.2021.2155>
102. Boyarsky, B.J. et al. Immunogenicity of a Single Dose of SARS-CoV-2 Messenger RNA Vaccine in Solid Organ Transplant Recipients. *JAMA*. 2021;325(17):1784–1786. <https://doi.org/10.1001/jama.2021.4385>
103. Grupper, A. et al. Reduced humoral response to mRNA SARS-CoV-2 BNT162b2 vaccine in kidney transplant recipients without prior exposure to the virus. *Am J Transplant*. 2021;00:1–8. <https://doi.org/10.1111/ajt.16615>
104. Chavarot, N. et al. Poor Anti-SARS-CoV-2 Humoral and T-cell Responses After 2 Injections of mRNA Vaccine in Kidney Transplant Recipients Treated with Belatacept. *Transplantation*: April 8, 2021 – Volume Online First – Issue – <https://doi.org/10.1097/TP.0000000000003784>
105. Peled, Y. et al. BNT162b2 vaccination in heart transplant recipients: Clinical experience and antibody response. *J Heart Lung Transplant*. 2021 Apr 21:S1053-2498(21)02274-9. <https://doi.org/10.1016/j.healun.2021.04.003>.
106. Rozen-Zvi, B. et al. Antibody response to SARS-CoV-2 mRNA vaccine among kidney transplant recipients: a prospective cohort study. *Clin Microbiol Infect*. 2021 May 3:S1198-743X(21)00216-0. <https://doi.org/10.1016/j.cmi.2021.04.028>
107. Agur, T. et al. Antibody response to mRNA SARS-CoV-2 vaccine among dialysis patients – a prospective cohort study. *Nephrol Dial Transplant*. 2021 Apr 11:gfab155. <https://doi.org/10.1093/ndt/gfab155>.
108. Grupper, A. et al. Humoral response to the Pfizer BNT162b2 vaccine in patients undergoing maintenance hemodialysis. *CJASN* Apr 2021, CJN.03500321; <https://doi.org/10.2215/CJN.03500321>
109. Torreggiani, M. et al. Neutralizing SARS-CoV-2 antibody response in dialysis patients after the first dose of the BNT162b2 mRNA COVID-19 vaccine: the war is far from being won. *Kidney Int*. 2021 Jun;99(6):1494-1496. <https://doi.org/10.1016/j.kint.2021.04.010>

110. Lacson, E. et al. Immunogenicity of SARS-CoV-2 Vaccine in Dialysis. medRxiv 2021 Apr 13:2021.04.08.21254779. <https://doi.org/10.1101/2021.04.08.21254779>
111. Anand, S. et al. Antibody Response to COVID-19 vaccination in Patients Receiving Dialysis. medRxiv 2021.05.06.21256768. 12 May. 2021, <https://doi.org/10.1101/2021.05.06.21256768>
112. Monin, L. et al. "Safety and immunogenicity of one versus two doses of the COVID-19 vaccine BNT162b2 for patients with cancer: interim analysis of a prospective observational study." *The Lancet. Oncology*, vol. 22,6 765–778. 27 Apr. 2021 [https://doi.org/10.1016/S1470-2045\(21\)00213-8](https://doi.org/10.1016/S1470-2045(21)00213-8)
113. Herishanu, Y. et al. Efficacy of the BNT162b2 mRNA COVID-19 Vaccine in Patients with Chronic Lymphocytic Leukemia. *Blood*, blood.2021011568. 16 Apr. 2021, <https://doi.org/10.1182/blood.2021011568>
114. Terpos, E. et al. Low Neutralizing Antibody Responses Against SARS-CoV-2 in Elderly Myeloma Patients After the First BNT162b2 Vaccine Dose. *Blood*, blood.2021011904. 16 Apr. 2021, <https://doi.org/10.1182/blood.2021011904>
115. Gavriatopoulou, M. et al. SARS-CoV-2 Vaccines in Patients With Multiple Myeloma. *HemaSphere* vol. 5,3 e547. 17 Feb. 2021, <https://doi.org/10.1097/HS9.0000000000000547>
116. Wong, S. et al. Serologic Response to Messenger RNA Coronavirus Disease 2019 Vaccines in Inflammatory Bowel Disease Patients Receiving Biologic Therapies. *Gastroenterology*, S0016-5085(21)00648-X. 20 Apr. 2021, <https://doi.org>
117. Kennedy, N.A. et al. Anti-SARS-CoV-2 antibody responses are attenuated in patients with IBD treated with infliximab. *Gut* vol. 70,5 (2021): 865-875. <https://doi.org/10.1136/gutjnl-2021-324388>
118. Sonani, B. et al. COVID-19 vaccination in immunocompromised patients. *Clin Rheumatol* 40, 797–798 (2021). <https://doi.org/10.1007/s10067-020-05547-w>
119. Wong, B.L.H. et al. Should children be vaccinated against COVID-19 now? *Archives of Disease in Childhood* Published Online First: 05 January 2021. <https://doi.org/10.1136/archdischild-2020-321225>

At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all made possible by digitalizing healthcare.

An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 120 years of experience and 18,000 patents globally. Through the dedication of more than 50,000 colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

All associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Published by

Siemens Healthcare Diagnostics Inc.
Laboratory Diagnostics
511 Benedict Avenue
Tarrytown, NY 10591-5005
USA
Phone: +1 914-631-8000